

Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



# Operator's Manual

## Shoulder/Knee Array Coil



### Shoulder/Knee Array – 16 CH RECEIVE-ONLY COIL


Revision 7 (02/10/2022)

FOR USE WITH GENERAL ELECTRIC  
1.5 or 3.0 TESLA 16-CHANNEL  
HD & HDx MR450 MR750 SYSTEMS ONLY

#### **Applicable Models:**

215GE1501 (sm/med)  
215GE3001 (sm/med)  
216GE1501 (large)  
216GE3001 (large)

Approved by:

  
Randall Jones, CEO/President

2-10-2022  
Date

Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



©2022 ScanMed, LLC

All rights reserved. No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system, or translated into any language in any form by any means without the written permission of ScanMed, LLC.

## LICENSES AND TRADEMARKS






The ScanMed® logo is a registered trademark of ScanMed, LLC.

Signa, Excite System, MR450, MR750 and the GE logo are registered trademarks of the General Electric Company.

Proper performance of this coil is warranted only on the system software for which it was specified at the time of purchase. Software or firmware upgrades may affect compatibility and performance. Please contact your GE representative and ScanMed Representative prior to operating on new software, as failure to do so may void your warranty.

**CAUTION:** Federal law restricts this device to sale, distribution, and use by or on the order of a physician.

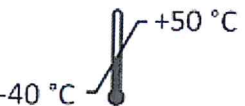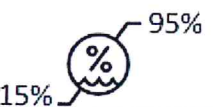

	Ambient temperature of -40 °C to +50 °C
	Relative humidity of 15% to 95% (non-condensing)
	Atmospheric pressure of 76.5 kPa to 101 kPa
	Protect from water
	Fragile, handle with care

## Table of Contents

LICENSES AND TRADEMARKS.....	2
EXPLANATION OF SYMBOLS .....	4
1.0 INTRODUCTION .....	5
1-1 Description .....	5
1-2 Applicable Models .....	5
1-3 Indications for Use .....	6
1-4 Accessories.....	6
2.0 INSTALLATION .....	7
2-1 Set-Up .....	7
3.0 OPERATION.....	8
3-1 Setup.....	8
3-2 Coil Selection .....	11
3-3 Landmarking and Imaging .....	11
3-4 Operation.....	11
3-5 Cleaning .....	11
4.0 QUALITY ASSURANCE.....	12
4-1 Quality Assurance Procedures .....	12
4-2 Initial QA Data.....	13
4-3 Data Table for Period QA Checks .....	14
5.0 SAFETY .....	15
5-1 General Safety .....	15
5-2 Contraindications and Patient Precautions.....	16
5-3 Cautions and Warnings .....	16
5-4 Emergency Procedures.....	17
5-5 Material Safety Information.....	17
6.0 MAINTENANCE, TROUBLESHOOTING, AND DISPOSAL .....	18
6-1 Preventative Maintenance and Cleaning .....	18
6-2 Troubleshooting .....	20
6-3 Disposal.....	21



Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



## EXPLANATION OF SYMBOLS

	Caution/warning, consult accompanying documents
	Attention, consult accompanying documents
	Type BF applied part
	Class II ordinary equipment, suitable for continuous operation
	For use on specified field strength
	Dispose of the coil by returning to manufacturer or via facility equipped to handle electronic products
	Part number and revision
	Serial number
	Manufacturer name, address, and date of manufacture
	<u>Authorized Representative</u> Emergo Europe Prinsessegracht 20 2514 AP The Hague The Netherlands
	<u>Medical Device Directive</u> These products conform to the requirements of council directive 93/42/EEC concerning medical devices, when they bear the following CE Mark of Conformity.

## 1.0 INTRODUCTION

### 1-1 Description

The shoulder/knee array interfaces with the G.E. 1.5T or 3.0T system and has been designed to collect image data throughout the region of the elbow, shoulder, knee, or ankle/foot depending upon patient and coil size.

This multi-channel design incorporates a set of unique antenna elements whose geometry has been optimized to image this anatomy. The coil form geometry has been formed to facilitate close coupling of the imaging coil's region-of-sensitivity to the anatomy of interest. The coil assembly comes with Velcro straps to comfortably place and secure the patient onto the coil assembly.



### 1-2 Applicable Models

This manual applies to the following models of the Shoulder/Knee Array Coil.

Model	MRI System Compatibility
215GE1501	1.5T GE HD or HDx,MR450
215GE3001	3.0T GE HD or HDx,MR750
216GE1501	1.5T GE HD or HDx,MR450
216GE3001	3.0T GE HD or HDx,MR750

Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



### 1-3 Indications for Use

The coil is indicated for use by the order of a physician to be used as an accessory to a GE 1.5T or 3.0T magnetic resonance scanner for general human anatomy imaging as supported by the scanner. These images, when interpreted by a trained physician, may assist in medical diagnosis.

### 1-4 Accessories

The following accessories may be purchased separately or as needed for replacement:

Part Number	Description
MSC00017F	Velcro Straps 58"
MSC00017C	Velcro Straps 22"

Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



## 2.0 INSTALLATION

### 2-1 Set-Up

The Shoulder Knee coil is plug-and-play and will be automatically recognized by the MRI system as one of the coil modes listed in Section 3. No other installation procedures are required.

Please follow the QA procedure in Section 4 to verify the coil is working properly and establish the baseline SNR specific to your system and setup.



## 3.0 OPERATION

### 3-1 Setup

**CAUTION:** Remove any other coil or unused accessory device from the magnet before using the coil. Unconnected coils may cause patient burns.



**CAUTION:** Do not attempt to scan with the coil disconnected or unplugged from the scanner. Patient burns may result.



For lower extremity imaging, the patient should be in the supine position, feet-first into the magnet. Position the coil so that the cable exit is pointed into the magnet.

The coil should always be operated with as tight of closure as possible. Simply wrap the coil about the elbow or lower extremity (see Figure 2) and Velcro snugly in place using the supplied short Velcro straps.



For upper extremity imaging, the patient should be in the supine position, head first. The coil is positioned about the shoulder or arm, and Velcro closed as snugly as possible (see Figure 1) using the supplied Velcro straps.

The longer Velcro straps are sized to wrap entirely around the patient's torso and hold the coil in place. Position the coil so that the cable exit is pointed into the magnet.



Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



The coil may be positioned about the patient's extremities in any manner in which it may be closed, provided that the anatomy is positioned parallel with the bore.


Run the cable in the most direct way to the coil connector port. Plug the connector to the MRI system connector port.

**IMPORTANT:** It is important to connect the quick-disconnect anterior and posterior cables to the correct GE system cable, or image quality will be compromised. The cable from the anterior section is colored blue and is to be connected to the system cable with the blue connector.



Connect the Shoulder/Knee coil to your GE MRI system using the figures below.

For 450, 450W, 750, and 750W, connect the coil to one of the scanner ports shown below on the left or right. Do not connect to the center port.

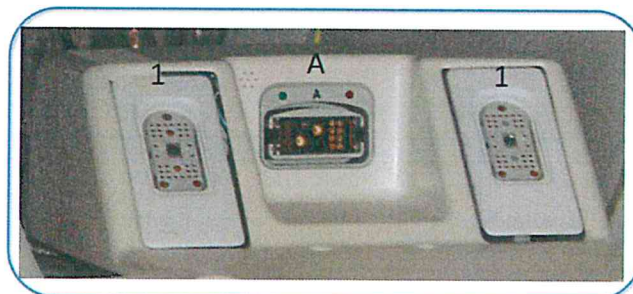

Scanner connections for 450/750.

Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



For 16-channel HDx systems:

1. For coils with two HDx connectors, connect both connectors to the A and B scanner ports shown below. Do not connect to the center port.
2. For coils with one HDx and one legacy connector, connect the HDx connector to B and the legacy connector to the center port. Do not connect to A.



Scanner connections for HDx.

**CAUTION!**



**Do not allow the cable to loop or contact the patient as this could create an RF burn hazard.**

**CAUTION!**



**Do not use the coil if tears are present in the foam covering or if the hard plastic casing is broken. Return the coil to the manufacturer for repair and/or replacement.**

**CAUTION!**



**Do not use the coil if the cable jacket is torn or ripped, or if metal is exposed. Return the coil to the manufacturer for repair and/or replacement.**

Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



### 3-2 Coil Selection

Use one of the following coil configurations depending on the model of your GE scanner. The entire coil is activated by these modes.

Use the JEXT configuration. The entire coil is activated by this mode.

Size	Model	Diameter	Length	Coil Modes
Medium	215GExxxx	110 mm	210 mm	Neocoil Anterior Array
Large	216GExxxx	180 mm	210 mm	Neocoil Anterior Array

**Coil Modes**

### 3-3 Landmarking and Imaging

Landmark on the anatomy positioned within the coil volume.

### 3-4 Operation

Note that the coil will provide imaging data throughout the entire length of the coil with proper coil selection. It is compatible with all array sequences.

### 3-5 Cleaning

If the coil is to be used for imaging only, only periodic routine cleaning is required per Section 6-1.



## 4.0 QUALITY ASSURANCE

### 4-1 Quality Assurance Procedures

Select a phantom to fill the volume of the coil. Perform an initial QA of the coil assembly. Run a standard T1-weighted sequence such as TR 500, TE 20, 5 mm coronal slice, with a 30 cm FOV. After the acquisition, record signal means (ROIs inside phantom) and noise standard deviations (ROIs outside phantom), as shown below.

Divide the ROIs inside the phantoms by the average of the ROIs outside the phantoms in each section. Record these numbers in table 4-2.

Refer back to these numbers and periodically repeat the measurement as a quality assurance test or if you suspect problems with the coil. If QA test results reveal SNR degradation of greater than 15%, call ScanMed™ customer service for further instructions.

Make additional copies of the following data tables as needed.




Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



## 4-2 Initial QA Data

ROI	Signal ROI Signal Mean	Noise ROI Standard Deviation	SNR (Mean/Average SD)
1			
2			
3			

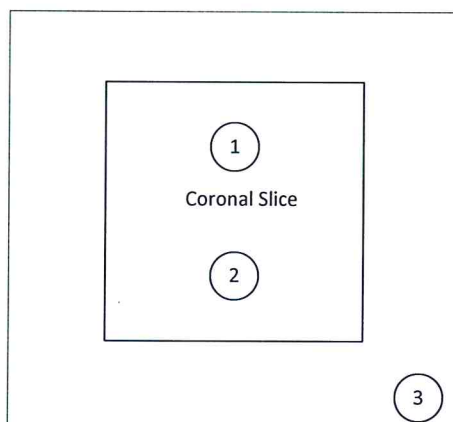

ROI placement. ROI's 1 and 2 are within the phantom volume, while ROI 3 is outside of the phantom volume.

Manufactured by:  
 ScanMed, LLC  
 9840 S. 140<sup>th</sup> St., Suite 8  
 Omaha, Nebraska 68138 USA



#### 4-3 Data Table for Period QA Checks

	Average SNR value	
Original Install		C

1	2	3
Date	Average SNR Value	Percent Deviation (Column 2 ÷ C)

## 5.0 SAFETY

### 5-1 General Safety

Patient safety and comfort must be your primary concern during the scanning procedure. Always follow proper safety procedures to ensure patient safety.

**CAUTION:** Remove any other coil or unused accessory device from the magnet before using the coil. Unconnected coils may cause patient burns.



**CAUTION:** Do not attempt to scan with the coil disconnected or unplugged from the scanner. Patient burns may result.



**CAUTION:** Do not allow the cable to loop or contact the patient as this could create an RF burn hazard.



Route cables in the most direct way possible, without forming loops. Place cables under a cushion whenever possible, and keep them from contacting the patient.

**CAUTION:** Do not use the coil if tears are present in the foam covering. Return the coil to the manufacturer for repair and/or replacement.



**CAUTION:** Do not use the coil if the cable jacket is torn or ripped, or if metal is exposed. Return the coil to the manufacturer for repair and/or replacement.



**WARNING:** No modification of this device is allowed. Do not modify this equipment with authorization from ScanMed.



## 5-2 Contraindications and Patient Precautions

**WARNING:** Do not scan patients who have MRI incompatible implants, metallic fragments, or other contraindications.



**Refer to your MRI system manufacturer's safety information.**

The operator should be aware of the following contraindications for use related to the strong magnetic field of the MR system:

- Scanning is contraindicated for patients who have electrically, magnetically, or mechanically activated implants (for example, cardiac pacemakers), because the magnetic and electromagnetic fields produced by the MR device may interfere with the operations of these devices.
- Scanning patients with intercranial aneurysm clips is contraindicated.

Precautions should be taken when scanning patients with the following conditions:

- A greater than normal potential for cardiac arrest.
- An increased likelihood for developing seizures or claustrophobia.
- Unconscious, heavily sedated, confused patients or those with whom no reliable communications can be maintained.

## 5-3 Cautions and Warnings

The following general caution statements apply to scanning with a magnetic resonance system. For further details, review the cautions/warnings included in your MR system operations manual.

- Cables should not be looped or crossed. Arcing and patient burns could result.
- Route all cables so that they do not contact the patient.
- Patients with ferromagnetic metal should not be scanned, because the magnetic field may interact with implanted surgical clips or other ferromagnetic materials.
- The safety of scanning fetuses has not been established.
- Persons with cardiac pacemakers or other implanted electronic devices should not enter the magnetic field delineated by the system's manufacturer.
- There is a risk to scanning feverish or decompensated cardiac patients.



- Facial makeup should be removed before scanning because many eye makeups contain metal flakes which can cause skin and eye irritation. Permanent eyeliner tattoos may cause eye irritation due to the presence of ferromagnetic particles.
- Patients who work in environments in which there is a risk of having embedded metallic fragments in or near the eye should be carefully screened before having an MR exam.

#### 5-4 Emergency Procedures

In the unlikely event that a coil creates smoke, sparks or makes an unusually loud noise or if the patient requires emergency assistance:

- Stop the scan if one is in progress.
- Disconnect the coil.
- Remove the coil from the patient.
- Remove the patient from the scan room if medical treatment is needed.

#### 5-5 Material Safety Information

**CAUTION:** This product may contain chemicals, including lead, known to the state of California to cause birth defects or other reproductive harm. Wash hands after handling.



## 6.0 MAINTENANCE, TROUBLESHOOTING, AND DISPOSAL

### 6-1 Preventative Maintenance and Cleaning

- **QA Test**

Perform a system quality assurance phantom test as outlined in Section 4 of this manual. If the values you obtain do not fall within normal operating parameters, then there may be a problem with the coil. Contact ScanMed<sup>™</sup> service department for assistance.

- **Storage**

Coils should be stored and used at the same room temperature as your MR system.

**CAUTION:** Do not store your coil at high temperatures (above room temperature) as damage to the foam or plastic may result.



- **Inspection**

Visually inspect the coil cover and cable assembly for cracks or missing insulation about the copper conductors of either assembly. Check the cable connector for secure fit within the coil housing and also check the electrical contacts on the cable end to ensure that they appear straight. Verifying that connection pins are not bent over before forcing a poor connection will ensure proper electrical connection and prevent further damage.

**CAUTION:** Do not use a damaged coil. Return the coil to the manufacture for repair if the coil is damaged.



## • Cleaning

The coil cover is not completely impermeable. Safeguards should be put in place to minimize its contamination when required. Surface cleaning of the material is the only action allowed using the solutions specified below. If the coil is damaged during cleaning, contact ScanMed for repair. This product contains no user replaceable or serviceable parts. Do not remove the coil cover as this will void your warranty.

The cleaning solutions listed below have been tested and are recommended for cleaning the coil(s) and pad(s). Spray or pour the cleaning liquid onto a soft cotton cloth and proceed to clean.

- Warm water: Safe for all areas of the coil or pads.
- Commercial dishwashing liquid solution (1oz/gallon of water): Safe for all areas of the coil.
- Alcohol solution (70% isopropyl / 30% water): Do not apply to adhesive backed materials such as labels, decals or Velcro fasteners.
- Cydex/Lysol: Do not apply to adhesive backed materials such as labels, decals or Velcro fasteners.

**CAUTION:** Do not spray or pour cleaning liquid directly onto the coil or cables. Apply to a soft cotton cloth and proceed to clean.



Manufactured by:  
ScanMed, LLC  
9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, Nebraska 68138 USA



## 6-2 Troubleshooting

The following is a list of common problems and solutions for those problems. If you cannot solve a problem by following the procedures in the manual, contact ScanMed™ between the hours of 7:30 AM and 5:30 PM (CST), Monday through Friday to arrange for service/repair.

### **ScanMed Customer Service**

9840 S. 140<sup>th</sup> St., Suite 8  
Omaha, NE 68138  
Tel: (402) 934-2650  
Fax: (402) 778-9699

### **Receiving No Signal**

**Problem:** You are scanning and yet receiving no signal.

- Solutions:**
1. Verify that you are transmitting with the body coil and receiving with the imaging coil.
  2. Verify that you have selected the “currently connected” or “JEXT” coil from the coil selection list.
  3. Verify that the cable is correctly connected to the system. The coil cable should be connected to the coil port.
  4. If all of the above check out and you still cannot get a signal, try to scan (transmit and receive) with the body coil. For this test, be sure to remove the imaging coil from the magnet bore before you scan with the body coil. If you still receive no signal the problem probably lies with the MR system. If the body coil scan is satisfactory, there is probably a problem with the ScanMed™ coil. Contact ScanMed™ for assistance.



## Image Quality

**Problem:** The SNR percentage obtained in the periodic quality assurance check is not greater than 85%, or the image quality is not what you expected it should be, given the parameters selected.

- Solutions:**
1. Review the selected protocol. The protocol should be identical to the recommended protocols provided.
  2. Verify that there are no loops in the cables.
  3. Verify that there are no metal or magnetic objects close to the coil, patient or magnet (e.g., safety pin, hair pin).
  4. Verify that the coil is properly positioned.
  5. Verify that your center frequency is within the frequency adjustment range for your system.

## Artifacts

**Problem:** There is a black line or signal void on the image (similar to an artifact seen when metal is present in the scanned area).

- Solutions:**
1. Verify that there is no metal present in the area being scanned.
  2. If the above checks out, it is possible the coil has failed. Call ScanMed<sup>™</sup>.

## 6-3 Disposal

Dispose of the coil by returning to the manufacturer or through a disposal facility equipped to handle electronic products.

**CAUTION: Dispose of RF coil properly.**

